
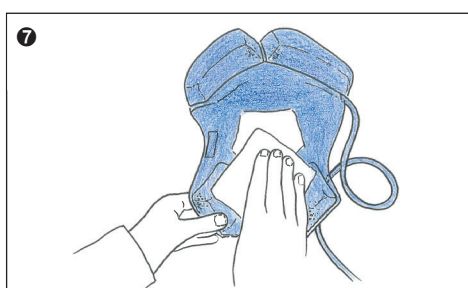
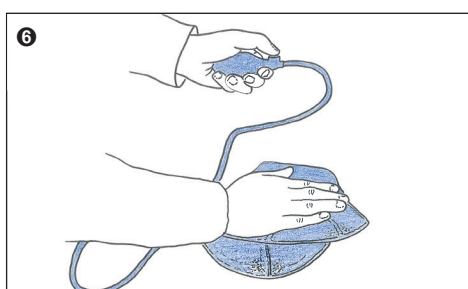
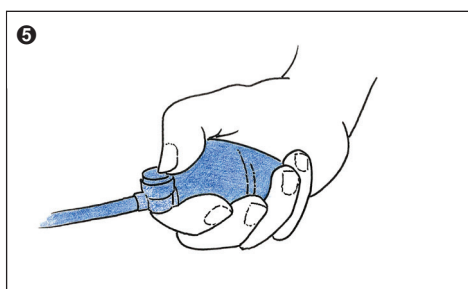
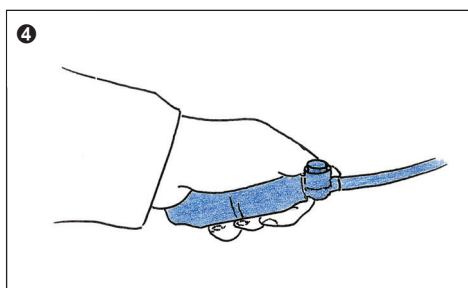
Crania

Instructions for Use
Gebrauchsanweisung
Mode d'emploi
Istruzioni per l'uso
使用説明書
Instrucciones de uso
Instruções de utilização
Brugsanvisning
Käyttöohje
Bruksanvisning
Gebruiksaanwijzing
คู่มือการใช้งาน
Инструкция по эксплуатации
Instrukcja obsługi
Notkunarleiðbeiningar
사용 설명서
使用説明
Kullanım Kitabı
Kasutusjuhend
Návod k použití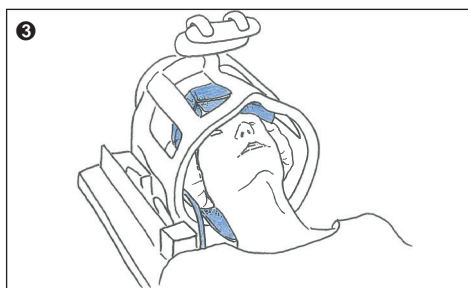

Head positioning aid for MR imaging

Short-term patient positioning and stabilization device during invasive and non-invasive procedures to be used on intact patient's skin. Non-Sterile and not to be used in the sterile field.

❶ Cover the patient's head with a single-use sanitary cover before use. The front edge of cover should be positioned just above the patient's eyebrows. Note: Using the cover is essential for sanitary reasons as well as to protect the positioning system.

❷ Place the disinfected Crania on the patient's head. The front edge of the Crania should be positioned just above the patient's eyebrows. If necessary, put on ear protection as usual.

❸ Move the patient into the correct position inside the head coil in preparation for the examination. Close the head coil.

❹ Pump up the Crania until the required degree of support has been attained. Do not over-pump. The patient should not find the pressure unpleasant. Position pump and tube to avoid them from getting caught when the patient is moved into the scanner. Position the pump far away from the field of view.

❺ Once the examination is complete (see instructions provided by the device manufacturer), deflate the Crania by pressing the valve on the hand pump. Open the head coil. Help the patient out of the head coil. Remove the Crania from the patient.

❻ Dispose of the single-use sanitary cover and squeeze the remaining air out of the Crania by hand.

❼ Clean the Crania using any readily available hospital disinfectant. The disinfectant should be applied as directed but thoroughly wiped off after application. Note: Do not immerse the Crania in cleaning fluid. Optimal functionality guaranteed until red indication line appears on the Crania.

Contraindications: Do not use the product with children below the age of 3. Do not use the product for purposes other than those described.

MR MR safe

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⚠ Do not sterilize. Do not allow direct contact with the skin. Use single-use sanitary cover. Do not use in case of acute open wounds. Do not inflate with compressed air. Position the pump far away from the field of view. Caution: Federal law restricts this device to sale by or on the order of a licensed health-care professional.

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