

CT HeadFix

Instructions for Use
Gebrauchsanweisung
Mode d'emploi
Istruzioni per l'uso
使用説明書
Instrucciones de uso
Instruções de utilização
Brugsanvisning
Käyttöohje
Bruksanvisning
Gebruiksaanwijzing
คู่มือการใช้งาน
Инструкция по эксплуатации
Instrukcja obsługi
Notkunarleiðbeiningar
사용 설명서
使用説明
Kullanım Kitabı
Kasutusjuhend
Návod k použití


1 A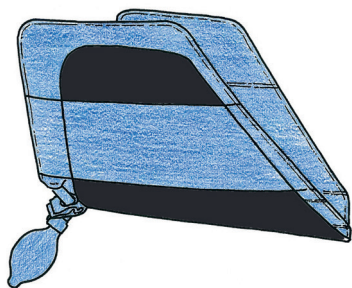


1 B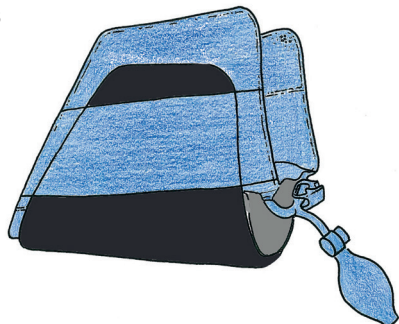


2

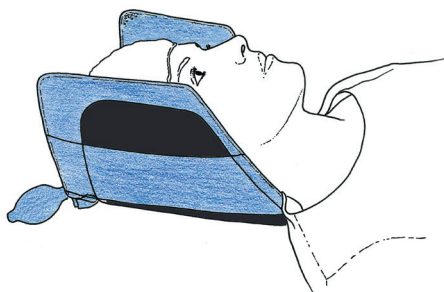
3

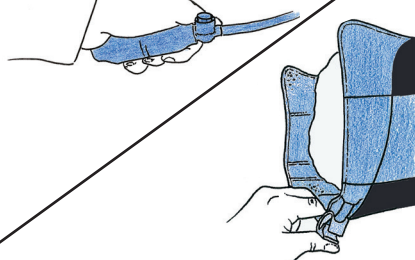4

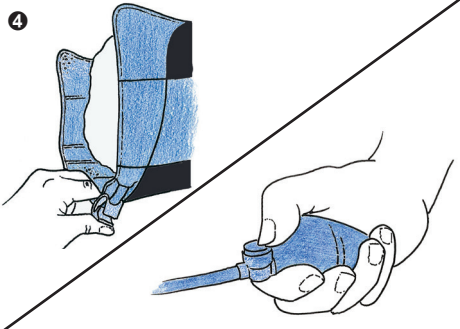5

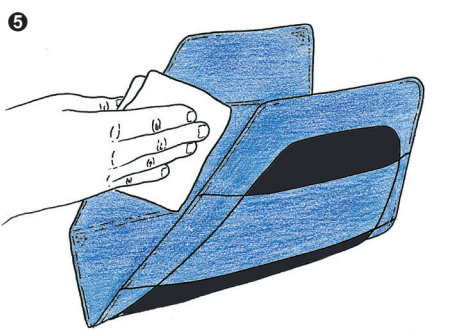Positioning aid for head fixation in the CT head holder

Short-term patient positioning and stabilization device during invasive and non-invasive procedures to be used on intact patient's skin. Non-Sterile and not to be used in the sterile field.

1 Version A (without inclination wedge): Put the cleaned CT HeadFix into the head holder and secure it using the side straps.

1 Version B (with inclination wedge): First put the inclination wedge into the head holder. Then put the cleaned CT HeadFix into the head holder and secure it with the side straps.

2 The head is placed into the head holder and positioned to prepare for measurement. A hygiene barrier (e.g. a head cover) must always be used for hygienic reasons and to protect the patient.

3 Inflate the CT HeadFix until the required immobilization is achieved. Do not over-inflate. The pressure applied must be adjusted individually to the patient's condition by qualified personnel. If required, close the hose clamp to prevent the exchange of air between the two air chambers.

4 Once the examination is complete (see the device manufacturer's instructions for use), undo the hose clamp, if necessary and deflate the CT HeadFix by pressing the valve on the hand pump.

5 Dispose of used hygiene barriers and wipe the CT HeadFix with any regular disinfectant. Do not immerse the CT HeadFix in cleaning fluid.

Contraindications: Do not use the product with children below the age of 3. Do not use the product for purposes other than those described.

Non sterile.

Do not sterilize. A hygiene barrier (mechanical barrier) must always be used. Do not inflate with compressed air. The pressure applied must be adjusted individually to the patient's condition by qualified personnel. Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare professional (Rx only).

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