
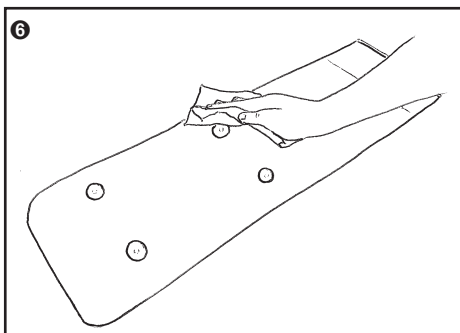
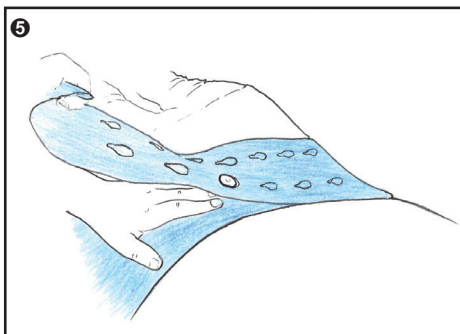
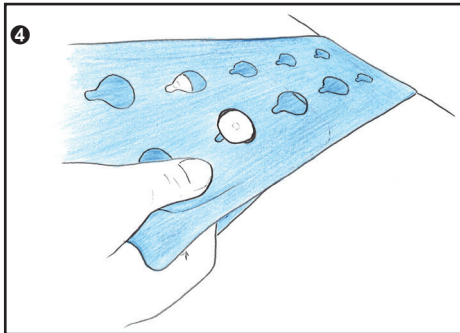
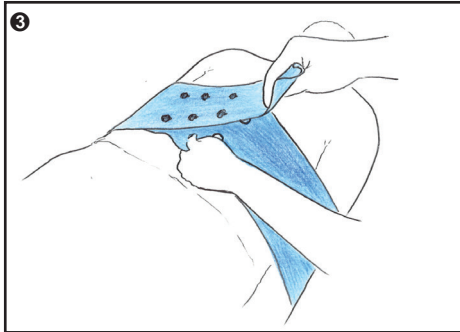
ProBelt

Instructions for Use
Gebrauchsanweisung
Mode d'emploi
Istruzioni per l'uso
使用説明書
Instrucciones de uso
Instruções de utilização
Brugsanvisning
Käyttöohje
Bruksanvisning
Användningsinstruktioner
Gebruiksaanwijzing
Инструкция по эксплуатации
Instrukcja obsługi
사용 설명서
使用説明
Návod k použití
تعليمات الاستخدام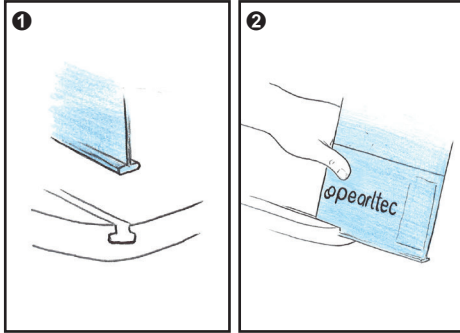

Rxonly **CE**

Federal law restricts this device to sale by or on order of a licensed healthcare professional (Rx only)

Universal patient positioning belt

Short-term patient positioning and stabilization device during invasive and non-invasive procedures to be used on intact patient's skin. Non-Sterile and not to be used in the sterile field.

Application

- ❶ Slide ProBelt into according table rail. Note: Wording „Pearltec“ can be read when ProBelt is closed.
- ❷ Slide the ProBelt to the area of the patient's anatomy that needs to be stabilized.
- ❸ Place both ends of the ProBelt on the patients body and tighten it according to the application's requirements.
- ❹ Insert button into eyelet of the tightened ProBelt to stabilize the patient. Only after the ProBelt has been closed the patient table may be moved to the starting position to begin the examination.
- ❺ After the examination, move the examination table back to the neutral position. Afterwards loosen the fixation by opening the lock and move the belt to the foot end of the examination table before the patient is released.
- ❻ Clean the ProBelt after every exam using commonly available hospital disinfectant. When not in use, place the ProBelt at the end of the table and lay both ends flat on top of each other to prevent the belt from getting caught when the table moves.

Contraindications: Do not use the product for purposes other than those described.

MR MR safe

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! Do not sterilize. Do not allow direct contact with the skin. Use single-use sanitary cover. Do not use in case of acute open wounds. Never move the examination table when ProBelt is hanging down from the side of the table.

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