

# Cleaning Instructions

## Pearltec Positioning Aids

This document provides a selection of cleaning and disinfectant solutions, which can be used to clean and disinfect Polyurethane foils of the following Pearltec positioning aids:

- MULTIPAD
- CT HeadFix
- CRANIA
- BabyFix Cocoon
- PearlFit
- PearlFit Vac
- ProFoam
- CT Table Cover
- ProBelt

## Recommended Cleaning and disinfectant solutions

Pearltec products can be cleaned and disinfected with any in the medical environment commonly used surface disinfectants, such as:

### Alcohol

- Incidin liquid (Ecolab)
- Mikrozyd AF liquid (Schülke)
- 70 % Alcohol (Hänseler)
- 2-Propanol (Sigma Aldrich)

### Non-Alcoholic

- Mikrozyd sensitive liquid (Schülke)

### Aldehyde-free

- Biguanid Fläche N (Schumacher / Dengg)
- TPH protect (Schülke)
- Kohrsolin FF (Hartmann)

### Oxygen releasing agents

- Perform (Schülke)

## Cleaning and disinfection instructions:

- A) Use a hygiene barrier between patient and Pearltec product for each application to additionally protect the surface of the product.
- B) Clean Pearltec product after each application with approved disinfectants for surface disinfection.
- C) Thoroughly clean and disinfect the entire product surface only by means of wiping, for example with prefabricated disinfection tissues. Do not spray/apply cleaning agent directly onto product. Do not sterilize products.
- D) ProFoam and ProBelt products (not ProBelt Thorax) can alternatively be immersed in cleaning bath.

