




# INSTALLING REMETRICHEALTH MOBILE APP

## INSTALLING REMETRICHEALTH MOBILE APP FOR ANDROID DEVICES

Follow these simple steps to download and get started with your RPM Medical Devices.

1. On an internet connected Android device, open the Google Play Store and in the Search field, enter RemetricHealth.
2. Tap on the Install Button
3. Once installed, tap the Open button
4. Enter the login details supplied by your provider
  - a. Email or phone number
  - b. Password
5. The dashboard is displayed, and you are ready to use your medical devices!
  - a. Confirm your readings on the app pop up

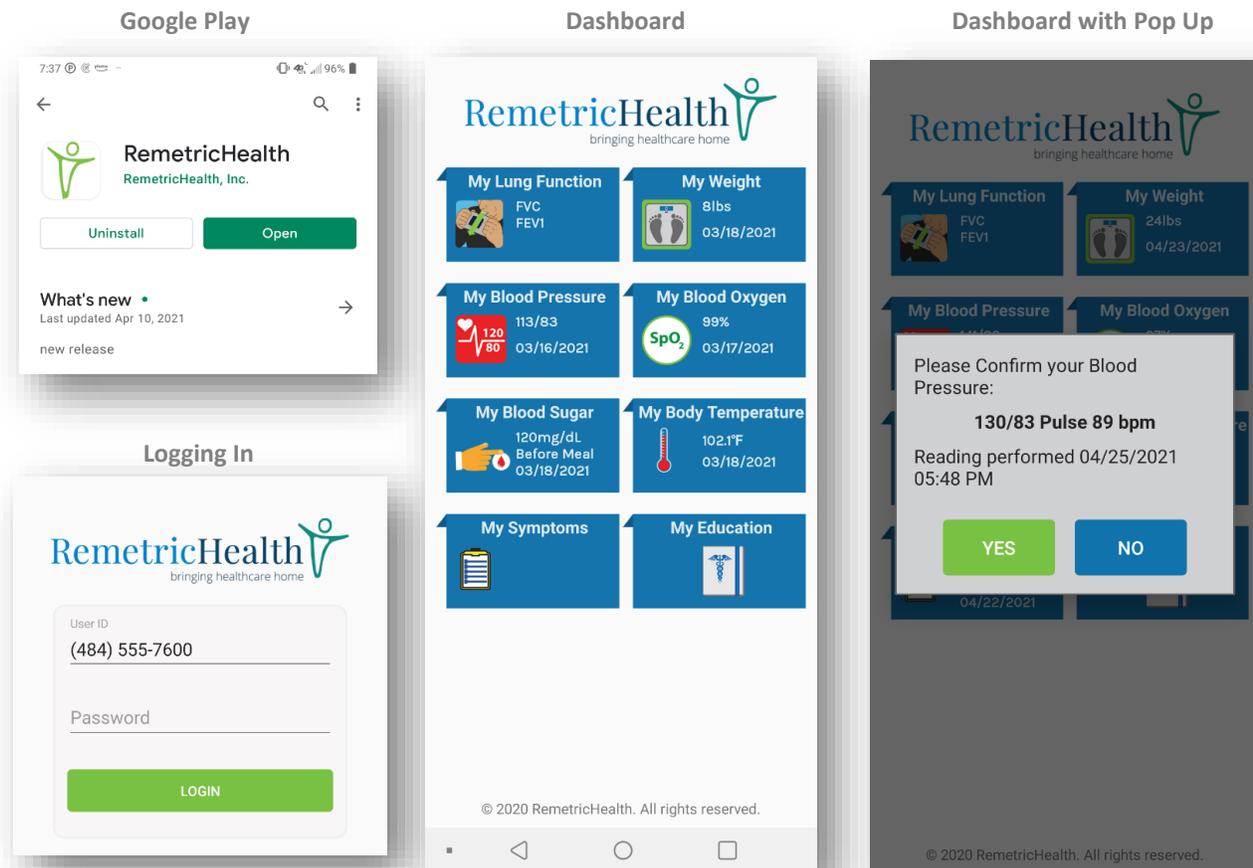

# RemetricHealth

## INSTALLING REMETRICHEALTH MOBILE APP FOR IOS DEVICES

Follow these simple steps to download and get started with your RPM Medical Devices.

1. On an internet connected Apple device, open the App Store and in the Search field, enter RemetricHealth.
2. Tap on the Install Button
3. Once installed, tap the Open button
4. Enter the login details supplied by your provider
  - a. Email or phone number
  - b. Password
5. The dashboard is displayed, and you are ready to use your medical devices!
  - a. Confirm your readings on the app pop up

