RemetricHealth

BIOMETRIC ALERTS

CREATING AN ALERT

 To create an alert for a patient, log into the patient profile and click on Settings

Account *			0 ; s	ettings L Log O	
D Healthcare			CARI-ANNE KELLE Last Log In: April 20th 20		
			ons 📉	5 TRI-TRENDS	
Biometrics Dashboard					
Biometric Measure	Me	easurement	Date	Time	
Spirometry	FI	FVC 5.16 FEV1 4 EV1/FVC 77%	03/23/2017	1:44pm	
Weight/BMI		174 lbs BMI 26.5	05/30/2019	1:08pm	
(SpO ₂) Oxygen Saturation (SpO2)			06/04/2019	7:14pm	

 Select the Biometric Alert you'd like to set



RemetricHealth

 Set the parameters and save the alert.
 Any measurements outside of these parameters will trigger an alert flag on the Provider Dashboard.

Biometric Alerts		You've a	dded a new alert user	
Therapy Alarms				
Medication	Sp02 Alert			
Notifications				
Patient's Profile	Alert If			
	Below		%	
		s	ave Changes	
	People Being N	Notified		
	Alert	Name	Email	SMS Number
	Sp02	Name	Email	999-999-9999

- 4. If you also want to send email or texts about this alert, complete the fields in the notification table at the bottom of the page and click the green plus sign to save.
- You'll receive an alert next time the patient has a reading outside of the parameter set.
 You can always go back into the patient's profile and adjust the alert as needed by following these steps again

If you have any questions, please contact customercare@remetrichealth.com.