



FRAME CLEANING GUIDELINES

The COVID-19 crisis is a stressful time for everyone, impacting each of us in our business and personal lives.

Your partners at Safilo have been working diligently to provide you guidance and information to implement "Frame Cleaning Guidelines" to sanitize your eyewear.

Safilo's preferred method of cleaning, which is 100% safe for all eyewear materials, is mild soap, free of scent and lotion, and clean running water. This cleaning process should be used for every frame being handled, or tried on, prior to returning it to your frame boards or storage. The first step to handling, and cleaning your eyewear, is to properly wash your hands, then apply properly fitting latex or vinyl gloves. The CDC.gov website recommends the following hand washing process. "Follow Five Steps to Wash Your Hands the Right Way"

FIVE STEPS to Wash Your HANDS the Right Way



FOLLOW THESE FIVE STEPS EVERY TIME

1. Wet your hands with clean, running water (warm or cold), turn off the tap, and apply soap.
2. Lather your hands by rubbing them together with the soap. Lather the backs of your hands, between your fingers, and under your nails.
3. Scrub your hands for at least 20 seconds. Need a timer? Hum the "Happy Birthday" song from beginning to end twice.
4. Rinse your hands well under clean, running water.
5. Dry your hands using a clean towel or air dry them.

STEPS to Clean Your EYEWEAR



Separate all frames that were handled or tried on by putting them aside in a clean, disinfected container, lined with a clean cloth or single use paper towel.

SOAP AND WATER PROCESS

1. Bring the container of handled eyewear over to your washing station.
2. Wet the frames with clean, running water, turn off the tap, and apply soap.
3. Lather the frames by gently rubbing with the soap, covering all areas of the frame parts.
4. Rub the frames with your finger tips for at least 20 seconds.
5. Rinse the frames well under clean, running water.
6. Dry the frames using a clean soft cloth.
7. Remove and discard latex or vinyl gloves.
8. Repeat hand washing process above before returning the frames to your frame boards or storage.

If you do not have access to perform the Soap and Water Process, we recommend using Hydrogen Peroxide Wipes, or a 50/50 solution of 3% Hydrogen Peroxide and water mixed in a spray bottle.

HYDROGEN PEROXIDE METHOD

1. Wipe down every frame from the container of handled eyewear.
2. Be sure to wipe every part of the frame, with gloved fingers, and a Hydrogen Peroxide Wipe.
3. Discard the used wipe, and do not reuse.
4. If wipes are not available, spray the Peroxide mixture on the frames, and wipe dry with a microfiber cleaning cloth.

If this process cannot be done right away, it is recommended to keep the frames separated, and not return them to your frame boards or storage until the cleaning process can be completed.