



# Oxivir Excel® Wipe CE Mini Guide

Available in 12x30 & 12x100



Ready to use for faster disinfection



Bactericidal and fully virucidal in 30 seconds



Oxivir Excel is certified as MDD - CE Class IIa medical devices (according the Medical Device Directive).

No Hazard classifications

## Use for disinfecting:




Reception area's



Lifts




Gym equipment

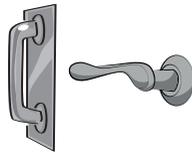


Door handles

Do not use on marble and aluminium

## Using...

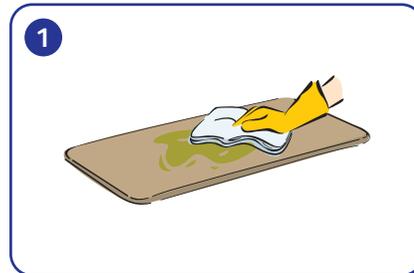


1 Remove excessive debris from the surface.

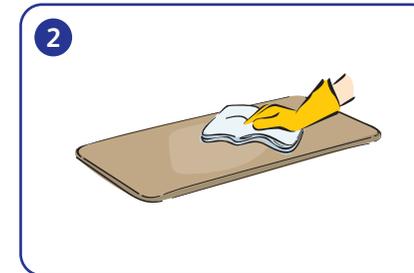

2 Clean and disinfect surface.



3 Allow 30 second contact time to meet virucidal claim according to EN14476.

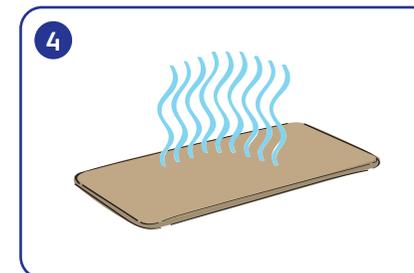

4 Allow to air dry.

