POCKET LASER PROCEDURE IN CASE OF IMPLANT UNCOVERING

Brief description of step by step procedure in case of implant uncovering using a 915 nm diode laser (Pocket Laser by 88Dent Pero-MI) with pre-loaded clinical programs and with optical fibers.

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Dott. Emanuele Ruga DDS; MSc

AIOLA Research Group

Figure 1 Inside SUREGERY section on Main Menu, some application are displayed .

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Figure 2 Implant Uncovering program offers to clinician two different modalities : STANDARD and SOFT.

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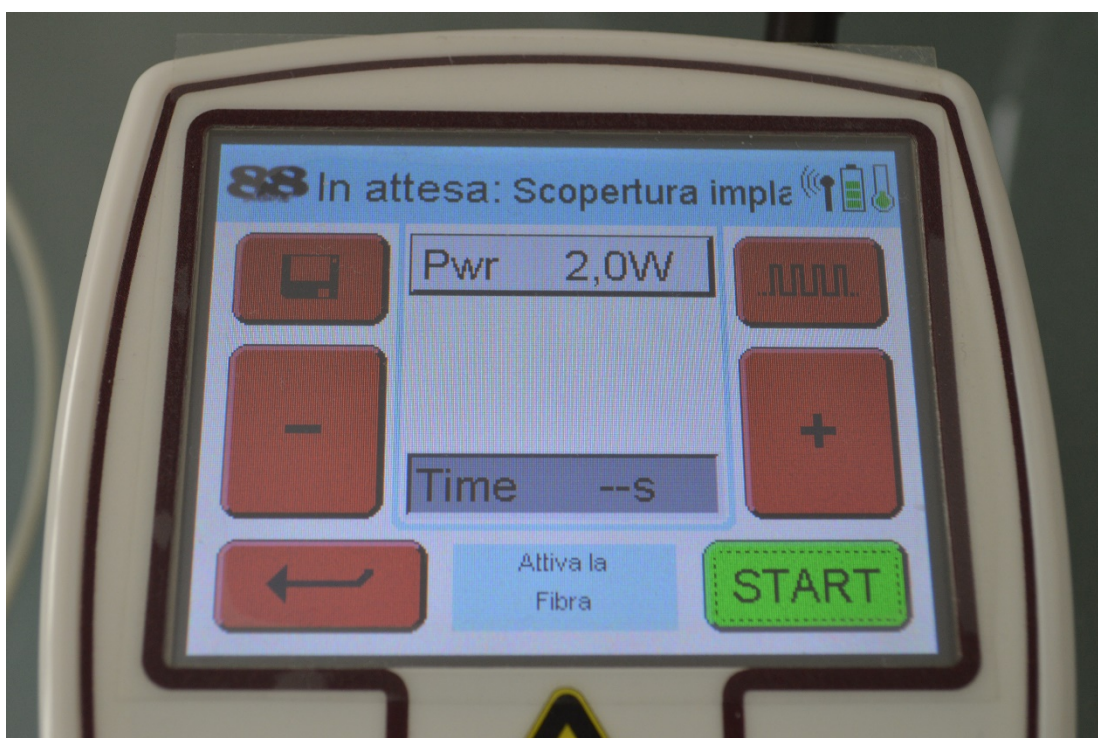
Figure 3 IMPLANT UNCOVERING PROGRAM: 2W CW, suitable for thick periodontal-gingival biotype and in case of local anaesthesia infiltration. Optical fiber initiation/activation is required.

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Figure 4 SOFT IMPLANT UNCOVERING PROGRAM ; PW 200 Hz 50 % Duty suitable in case of topical anaesthesia(or without anaesthesia) and thin periodontal-gingival biotype. Optical fiber initiation/activation is required.

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Figure 5 Either 200; 300 or 400 μ m optical fibers can be used. During procedure a counter shows on display emission mode, power applied and time required.

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Figure 6 Pre-operative intraoral view shows an implant site four months after fixture positioning. Head of implant fixture can be observed through a medium thickness gum transparency.

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Figure 7 Preoperative RX endoral exam shows osseointegration of a Conical shape 3.8mm x11,5 mm implant fixture in 2.5 position

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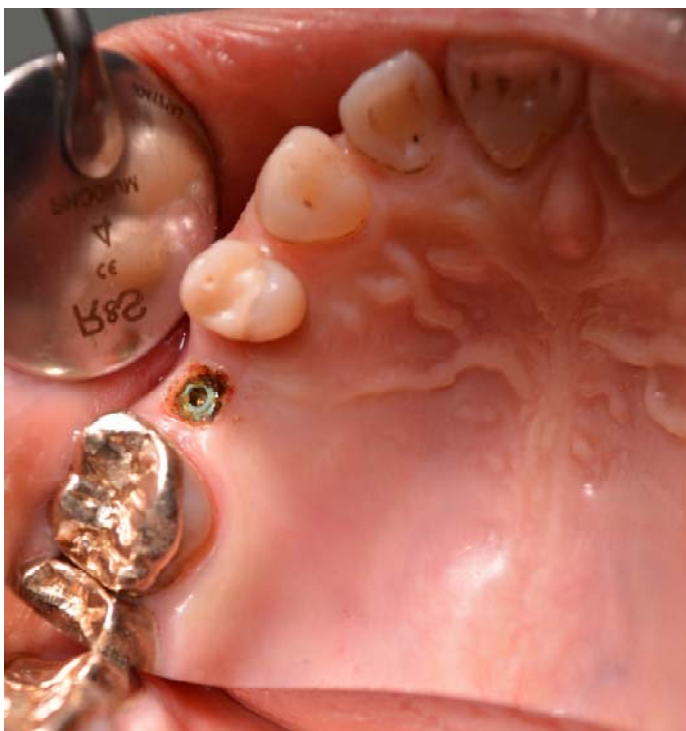
Figure 8 Surgeon performs a gentle operculization moving a 400 µm optical (initiated/activated) fiber in contact mode on soft tissue. Implant uncovering procedure is performed in less than 20 seconds without local anaesthesia. A gentle air flow is used to decrease temperature developing and to avoid undesired thermal damage.

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Figure 9 Postoperative endoral view shows the absence of bleeding or swelling and the correct positioning of healing abutment. Patient referred absence of pain perception during procedure and post procedure.

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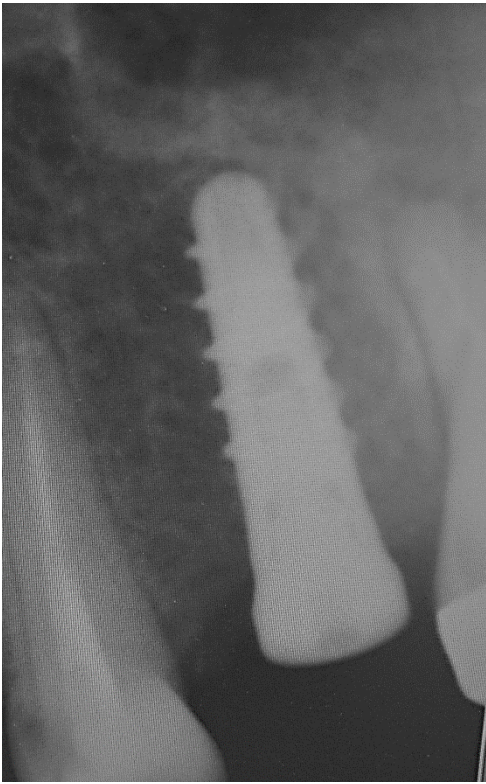
Figure 10 Postoperative RX endoral exam shows the correct positioning of healing abutment.

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