



## QUICK PAIRING GUIDE

# Pairing the Arkray GLUCOCARD Shine Connex Glucometer with the MyLife365 Tablet and Phone Hub



Follow Manufacturer Instructions for Use.

*Still have questions or need assistance? Contact Support at [support@life365.health](mailto:support@life365.health)*


# CONTENTS

Pairing the Arkray GLUCOCARD Shine Connex Glucometer with the MyLife365 <b>Tablet</b> .....	1
Pairing the Arkray GLUCOCARD Shine Connex Glucometer with the MyLife365 <b>Phone Hub</b> .....	3
Manual Pairing .....	7

# Pairing the Arkray GLUCOCARD Shine Connex Glucometer with the MyLife365 Tablet



- 1 After assigning a smart device to a member, select the Reminders application. You will receive two prompts, select “**ALLOW**” for both prompts.

**This step must be completed before pairing Bluetooth Devices.**

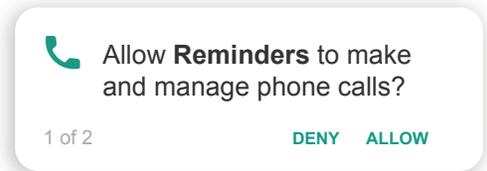



- 2 Select the Bluetooth Icon\*.  
Enter the password “**3651**” and select “**OK**”.

*\*the Bluetooth App Icon location may vary by device*

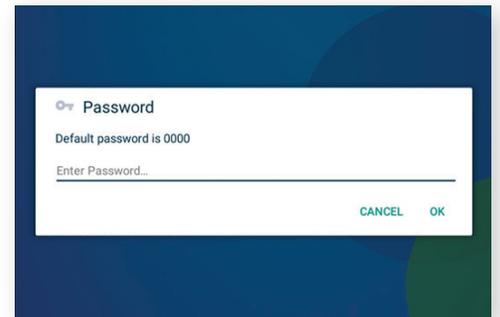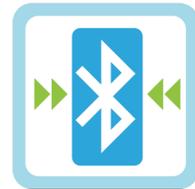

- 3 On the “**My Devices**” page, select “**Add New**” (A).

The smart device will search for the Bluetooth device.

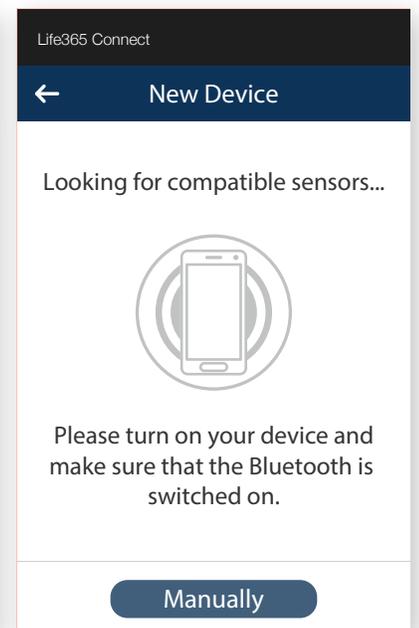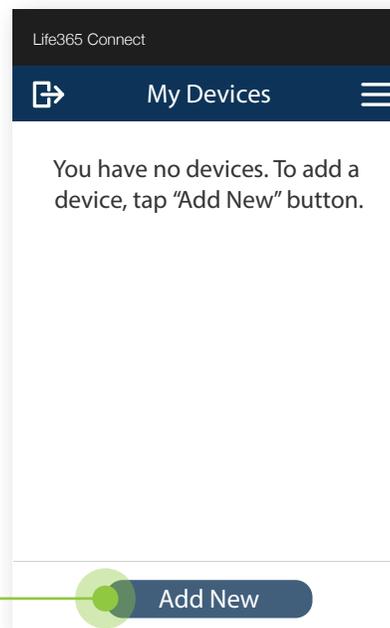

# Pairing the Arkray GLUCOCARD Shine Connex Glucometer with the MyLife365 Tablet

4 Turn **ON** (D) the Arkray GLUCOCARD Glucose Meter by selecting the power button **ONCE**.

5 Hold the power button for 3-4 seconds until the “**SET**” screen displays. Press the right arrow button **ONCE** until “**YES**” displays on the Glucose meter screen. Press the power button **ONCE** to proceed.

6 On the screen “**BT**”, press the right arrow button 3-times until “**PAIR**” displays (E) on the Glucose meter screen. Press the power button **ONCE** to proceed with pairing.




7 Once found, select the Bluetooth device name (D).

Select “**Add to My Devices**” (E).

*This is an example. Device name displayed will reflect the Bluetooth device the user is attempting to pair.*


8 Device pairing is now complete.

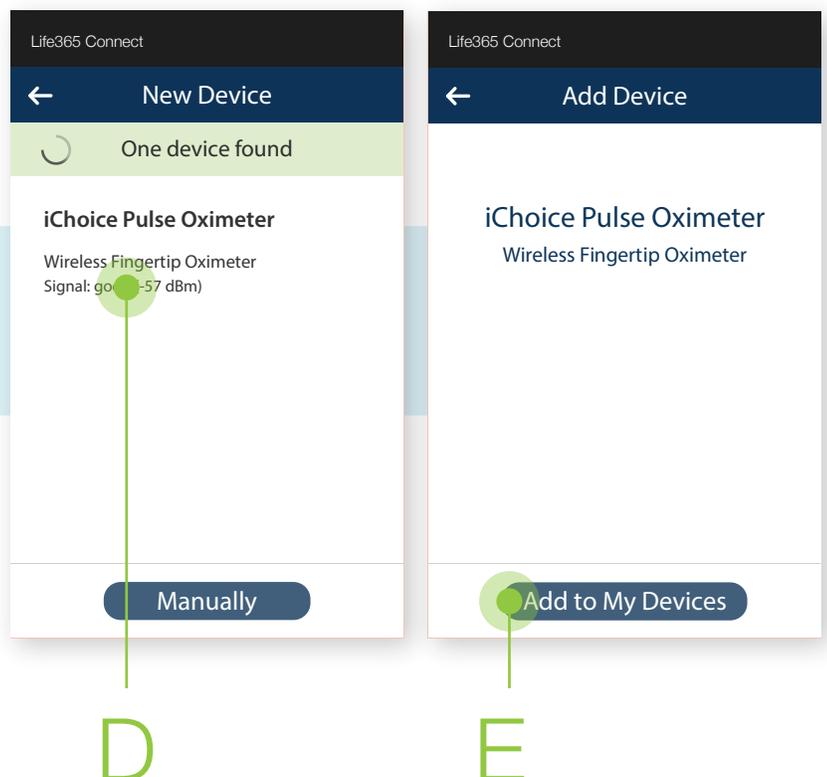

# Pairing the Arkray GLUCOCARD Shine Connex Glucometer with the MyLife365 Phone Hub



- 1 After assigning a smart device to a member, select the Reminders application. You will receive two prompts, select “**ALLOW**” for both prompts.

This step must be completed before pairing Bluetooth Devices.

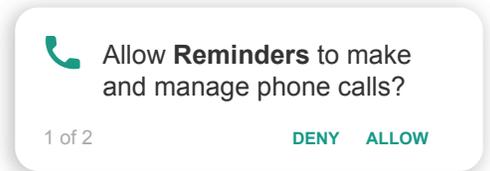

- 2 Select the Bluetooth Icon\*.  
Enter the password “**3651**” and select “**OK**”.


*\*the Bluetooth App Icon location may vary by device*

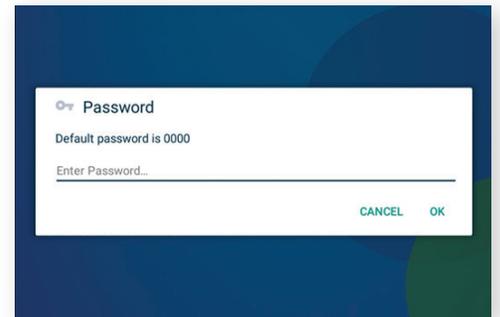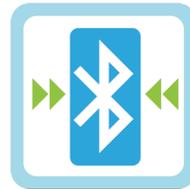

- 3 Within the **Admin Settings**, scroll downwards on the screen until the option **Exit SureLock (A)** appears.

Select this option, then Select **Exit**.

A

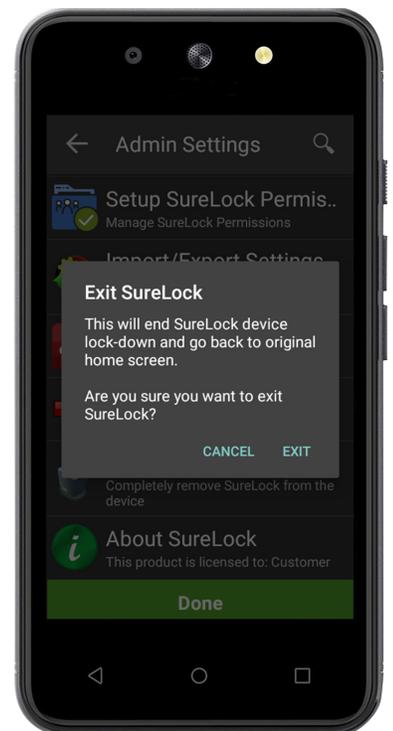

# Pairing the Arkray GLUCOCARD Shine Connex Glucometer with the MyLife365 Phone Hub

- 4 Select the **white round button (B)** located in the bottom center of the Hub's screen.

Navigate to and Select the application **Life365 Connect (C)**.



- 5 Turn **ON (D)** the Arkray GLUCOCARD Glucose Meter by selecting the power button **ONCE**.

- 6 Hold the power button for 3-4 seconds until the **"SET"** screen displays. Press the right arrow button **ONCE** until **"YES"** displays on the Glucose meter screen. Press the power button **ONCE** to proceed.

- 7 On the screen **"BT"**, press the right arrow button 3-times until **"PAIR"** displays (E) on the Glucose meter screen. Press the power button **ONCE** to proceed with pairing.



# Pairing the Arkray GLUCOCARD Shine Connex Glucometer with the MyLife365 Phone Hub


8

On the Hub, Select **Add New**. The Hub will search for the Bluetooth device.

Select the found Bluetooth device on the Hub's screen (F).

“**Keep history on device**” will display. Select “**Yes**” (G).

Select Add to my Devices.

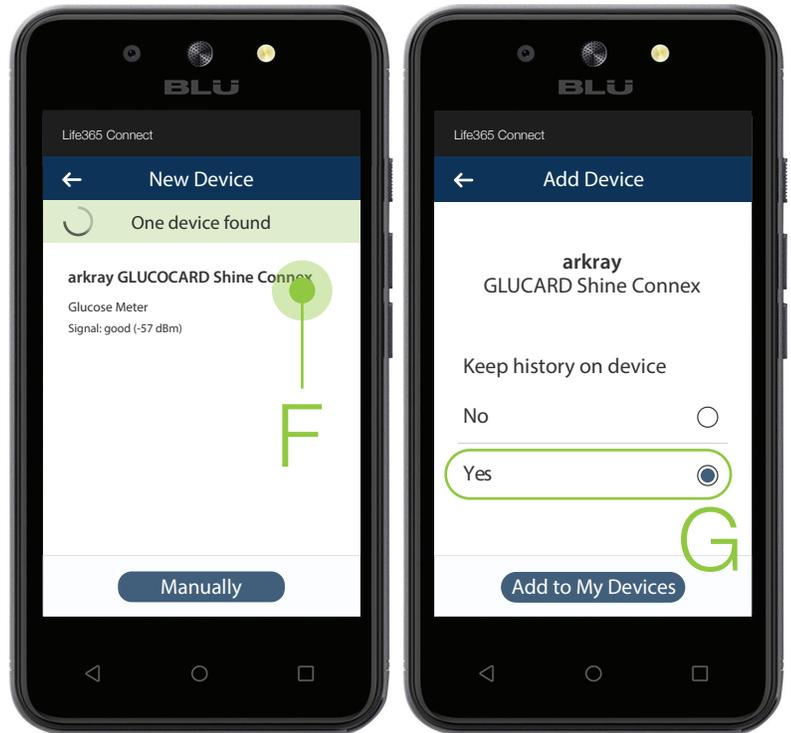


9

The Hub will chime. With an index finger, slide downwards from the navigation bar located at the top of the Hub screen to reveal notifications.

Select the notification **Pairing Request**.

Enter the **6-digit code** displayed on the Glucose meter screen. Check the box “**Allow access to your contacts & call history**”. Select “**OK**”.

Glucose meter will display “**Success**” on successful pairing. Turn **OFF** the Glucose meter by pressing the power button **ONCE**.

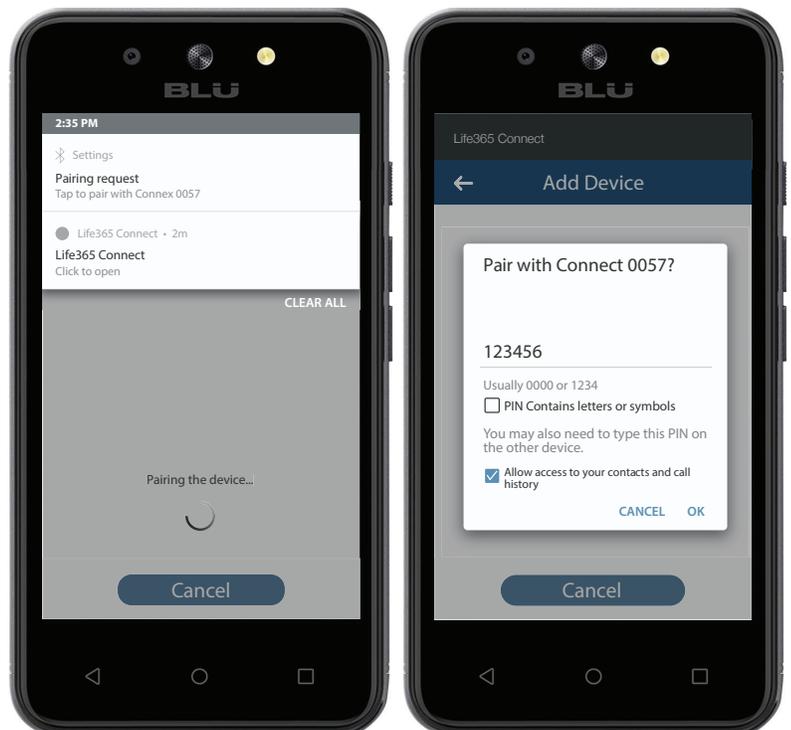

# Pairing the Arkray GLUCOCARD Shine Connex Glucometer with the MyLife365 Phone Hub

- 10 To re-enter Kiosk mode, Select the **Home** button to Exit the Life365 Connect application.

Select the **white round button (H)** located in the bottom center of the Hub's screen.

Navigate to and Select the application **SureLock (I)**.



- 11 You have successfully paired the Arkray GLUCOCARD Glucose Meter and entered Kiosk mode. Select the **Bluetooth Pairing** application to continue pairing additional Bluetooth devices.





# MANUAL PAIRING:

## Difficulty pairing this device?

- 1 Try manually pairing the device by selecting “**Add New**” on the “**My Devices**” page (A), then selecting “**Manually**” (B).
- 2 Search for the device by Manufacturer by selecting the magnifying glass (C).
- 3 Select the device, then select “**Add to My Devices**”.

Device pairing is now complete.

Take a test reading with the device.  
Hub will announce the reading.

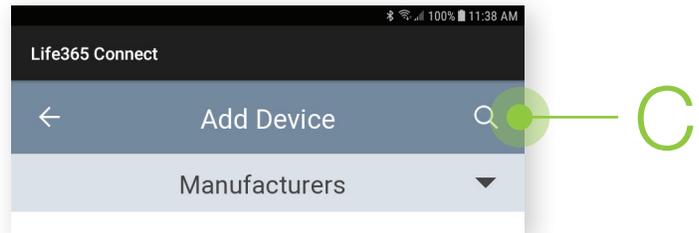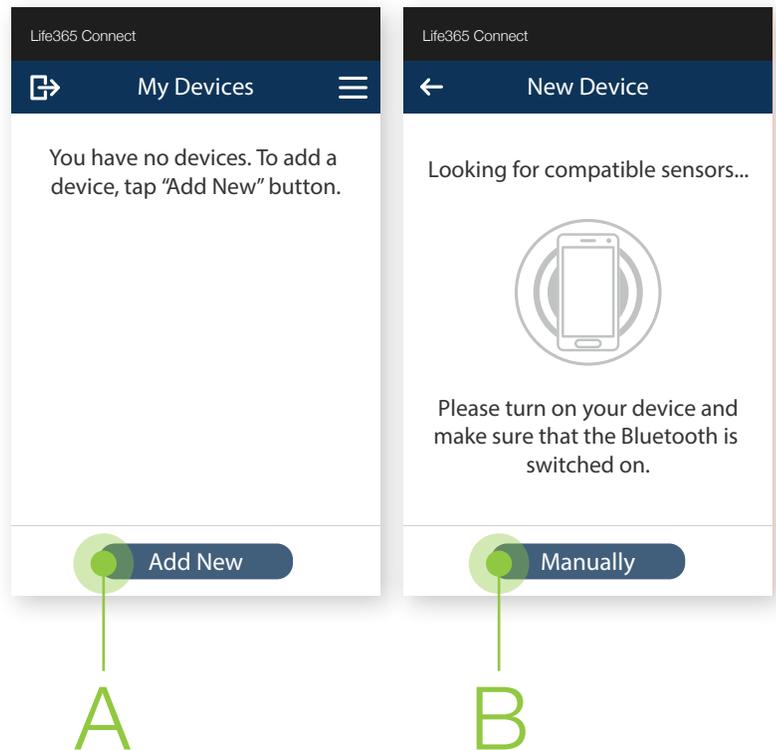


### NOTE:

The Reminders Application must be opened BEFORE opening the Bluetooth Application. If the Bluetooth Application is opened before the Reminders Application, a “**Choose Server Location**” error screen will display.

If this error screen is shown:

1. Exit the Bluetooth Application
2. Enter the Reminders Application
3. Accept both prompts
4. Re-enter the Bluetooth Application
5. Select the Refresh symbol (D).

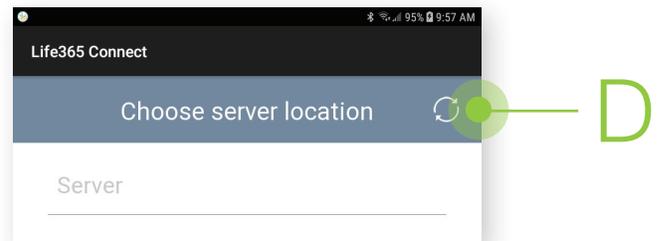

**You can now complete device pairing.**