


Pairing the A&D Blood Pressure Monitor with the Life365 App

 Follow Manufacturer Instructions for Use.
For additional assistance, contact your Care Provider.

Version Update August 2020

- 1 Select the **Hamburger Icon** (), located in the upper left of the Life365 app screen. Select **My Devices**.
- 2 On the My Devices screen, select the button **Add New**. The smart device will search for your Bluetooth device.
- 3 To pair, press the **START** button for 3-4 seconds, until “**PR**” displays on the blood pressure monitor screen.
- 4 Once found in the app, select the Bluetooth device **name**.

*If more than one A&D Medical blood pressure monitor is found, select the name of the device with the accurate model number. For example, **A&D UA-651BLE**.*

- 5 Select the button **Add to My Devices**.

Device pairing is now complete.

For video instructions, and other helpful resources, visit
<https://www.life365.health/en/support/app>.