

Pairing the Trividia Glucose Meter with the Life365 App

 Follow Manufacturer Instructions for Use.
For additional assistance, contact your Care Provider.

Version Update August 2020

- 1 Select the **Hamburger Icon (≡)**, located in the upper left of the Life365 app screen. Select **My Devices**.
- 2 On the My Devices screen, select the button **Add New**. The smart device will search for your Bluetooth device.
- 3 To pair, take a test blood glucose reading.

After the blood glucose reading displays on the meter, eject the test strip. The glucose meter will then turn **OFF**.

Turn the device back **ON** by pressing the oval, middle power button located on the top of the device.

- 4 Once found in the app, select the Bluetooth device **name**.
- 5 Select the button **Add to My Devices**.
- 6 A prompt may display requesting a code to complete device pairing. If a prompt does not display, complete the following:

With an index finger, swipe downwards from the top of your smart device screen to reveal Notifications.

Select the notification **Pairing Request**.

Enter the last **6-digits** of the serial number, located on the back of the Bluetooth device.

Check the box **“Allow access to your contacts and call history”**. Select **OK** once completed.

Device pairing is now complete.

For video instructions, and other helpful resources, visit
<https://www.life365.health/en/support/app>.