


Pairing the Arkray Glucose Meter with the Life365 App


 Follow Manufacturer Instructions for Use.
For additional assistance, contact your Care Provider.

Version Update August 2020

- 1 Select the **Hamburger Icon (A)**, located in the upper left of the Life365 app screen. Select **My Devices**.

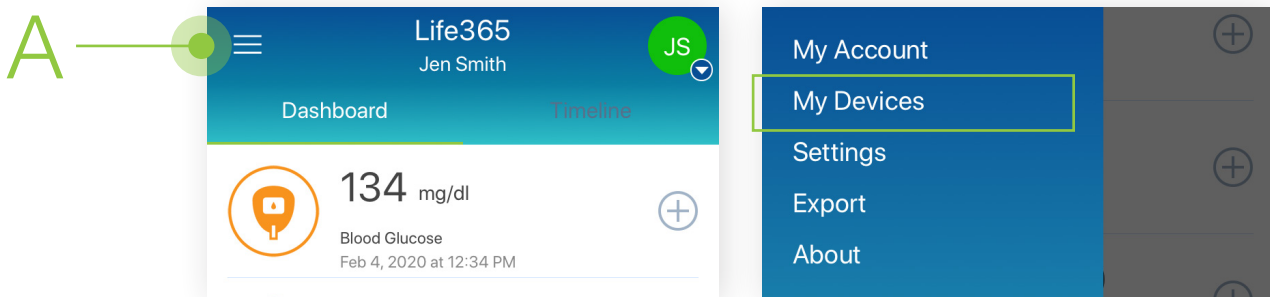

- 2 On the My Devices screen, select the button **Add New (B)**. The smart device will search for your Bluetooth device.

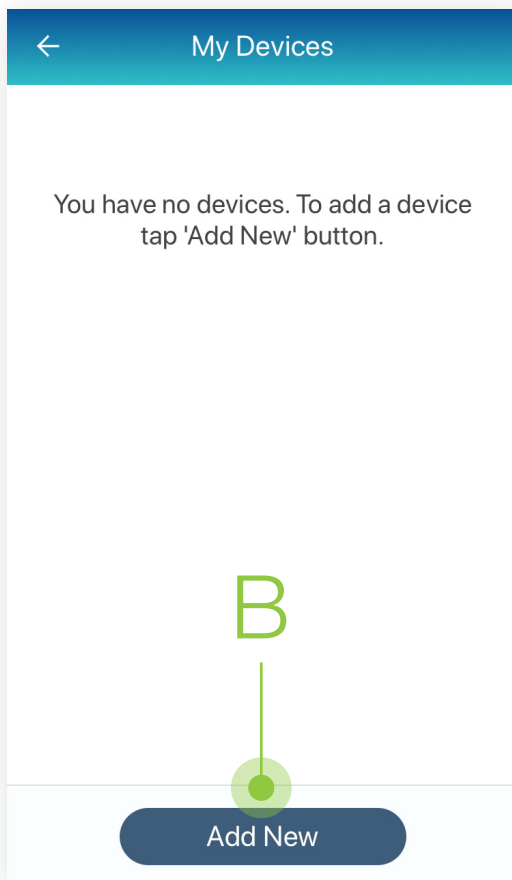

- 3 Turn **ON** the Arkray GLUCOCARD Glucose Meter by selecting the power button (C) **ONCE**.

Hold the power button for 3-4 seconds until the **"SET"** screen displays. Press the right arrow button **ONCE** to proceed.

On the screen **"BT"**, press the right arrow button 3-times until **"PAIR"** displays on the glucose meter screen (D). Press the power button **ONCE** to proceed with pairing.





Pairing the Arkray Glucose Meter with the Life365 App

- 4 Once found in the app, select the Bluetooth device **name (E)**.

On the screen **Keep History on Device**, tap **Yes (F)**.

Select the button **Add to My Devices (G)**.

This is an example. Device name displayed will reflect the Bluetooth device the user is attempting to pair.

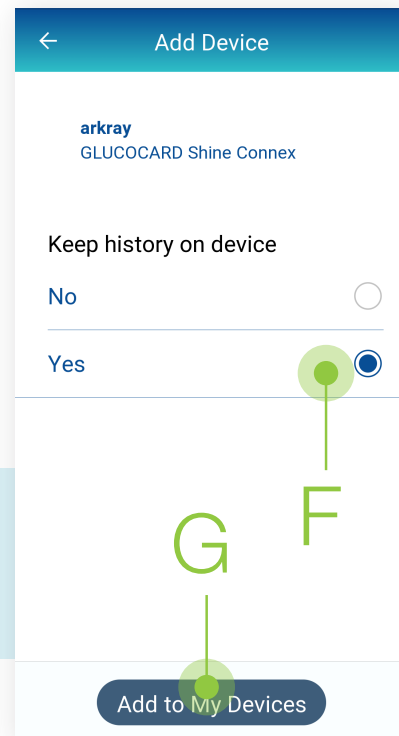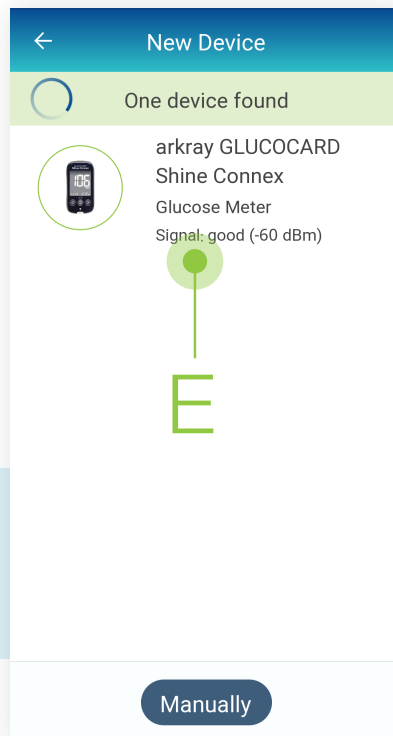


- 5 A prompt may display requesting a code to complete device pairing. If a prompt does not display, complete the following:

With an index finger, swipe downwards from the top of your smart device screen to reveal Notifications. Select the notification **Pairing Request**.

Enter the **6-digit code** displayed on the Glucose meter screen **(H)**.

Check the box **“Allow access to your contacts and call history” (I)**. Select **OK** once completed.

The Glucose meter will display **“Success” (J)** on successful pairing. Turn **OFF** the Glucose meter by holding the power button **(C)** for **3-4 seconds**.

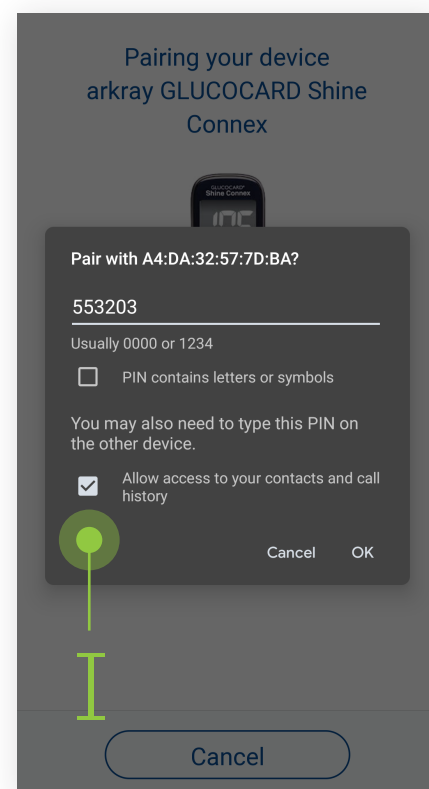

Device pairing is now complete.

For video instructions, and other helpful resources, visit <https://www.life365.health/en/support/app>.