


Pairing the TaiDoc/Fora Thermometer with the Life365 App


 Follow Manufacturer Instructions for Use.
For additional assistance, contact your Care Provider.

Version Update August 2020

- 1 Select the **Hamburger Icon (A)**, located in the upper left of the Life365 app screen. Select **My Devices**.

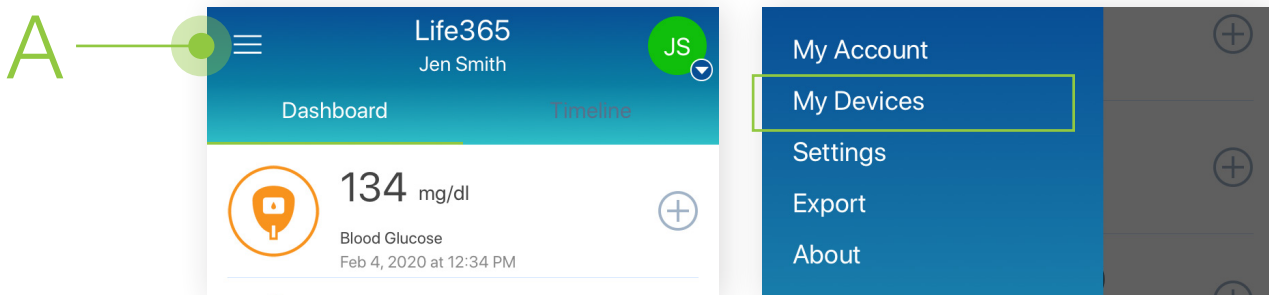

- 2 On the My Devices screen, select the button **Add New (B)**. The smart device will search for your Bluetooth device.

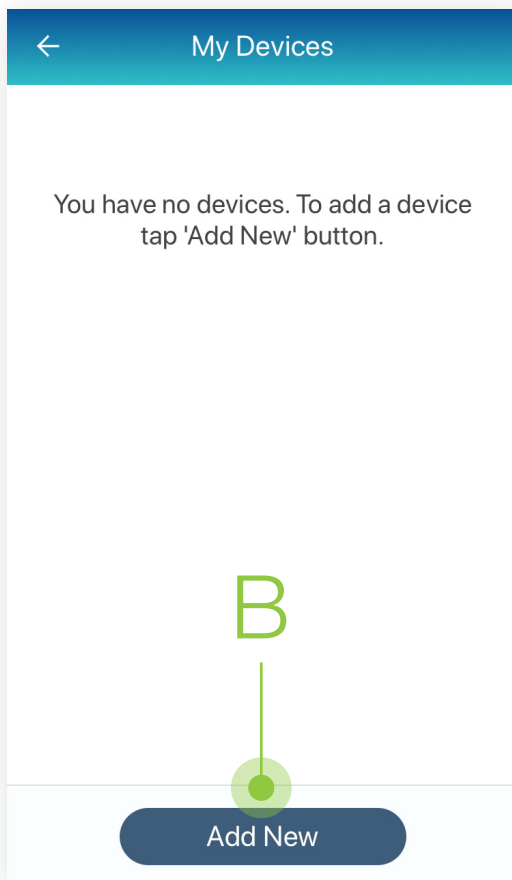

- 3 The TaiDoc/Fora Thermometer requires a measurement reading to complete initial pairing:

3.1 Insert an ear protection cover on the metal tip (C) of the thermometer.

3.2 Press the power button (D) to turn **ON** the thermometer.

3.3 Take a measurement reading with the thermometer (E).





Pairing the TaiDoc/Fora Thermometer with the Life365 App

- 4 Once found in the app, select the Bluetooth device **name (F)**.

Select the button **Add to My Devices (G)**.

This is an example. Device name displayed will reflect the Bluetooth device the user is attempting to pair.

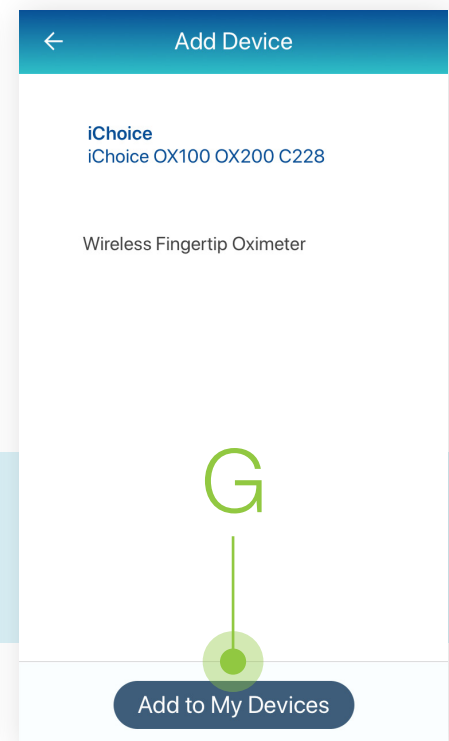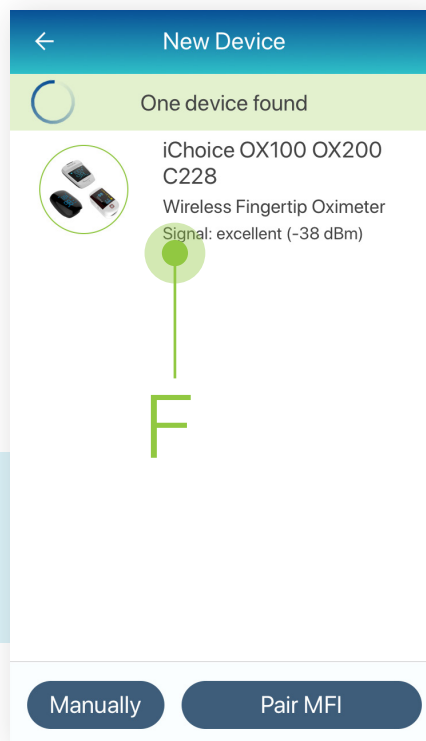

Device pairing is now complete.

For video instructions, and other helpful resources, visit <https://www.life365.health/en/support/app>.