

Your Guide to MyLife365 Applications



2-Way Video



Camera



Cardiac Survey



Resource Library



Reminders



Resource Library



Reminders




*for the MyLife365
Tablet and Phone Hub
Smart Devices

Reminder Application



Create Reminders for your Members to receive on their Smart Device.

Reminder messages will display on the screen when Smart Device is in sleep-mode.

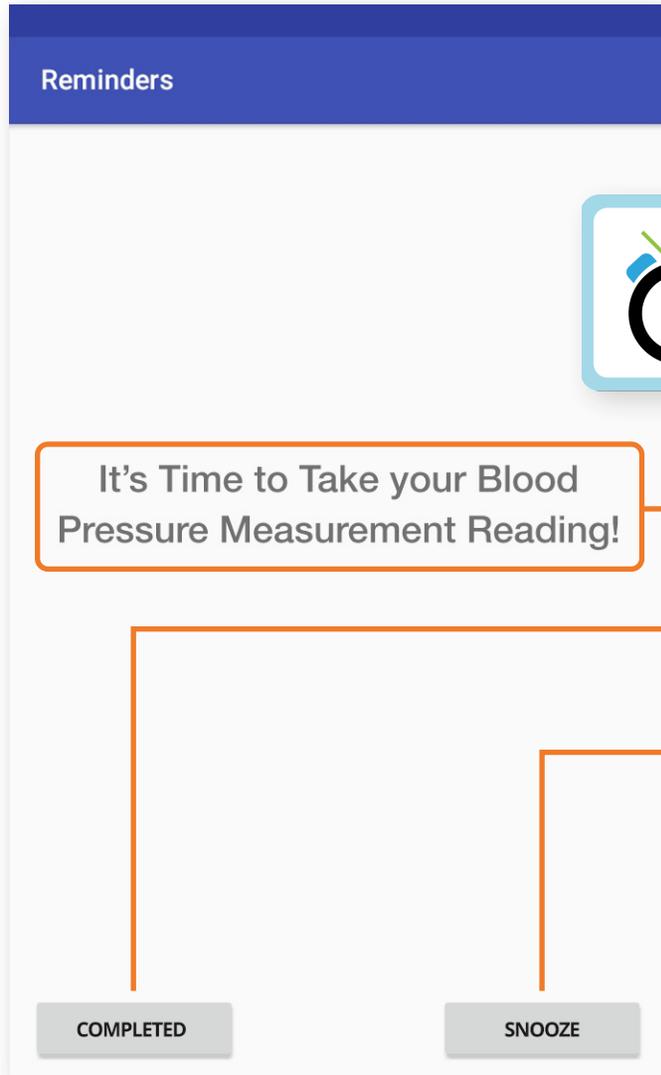

Open the Reminder application.

View Reminder message on the screen.

Select Completed to complete Reminder.


Select Snooze to delay reminder for 3-minutes.

Note you can only snooze a reminder three times before it's recorded as Missed.

In some instances, the application may require you to allow premissions.

Accept these prompts. The application will then launch.

Survey Application

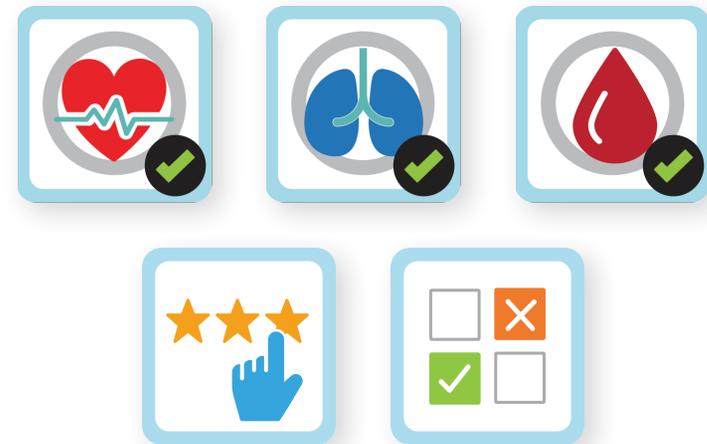

Select a **survey icon** to complete a survey.

All survey responses are sent to the Coordinator Portal for review.

MyLife365 Patient Portal



Open the Patient Portal application to access:

Vitals Data
Personal Information
Care Coordinator Information

Resource Library


Open the Resource Library application to access:

- Quick-Tip Guides
- Web-Links
- MyLife365 Tablet Application Guides

Select the appropriate folder to access resource materials.

Select to open the PDF or HTML file to view.


Bluetooth Pairing Application

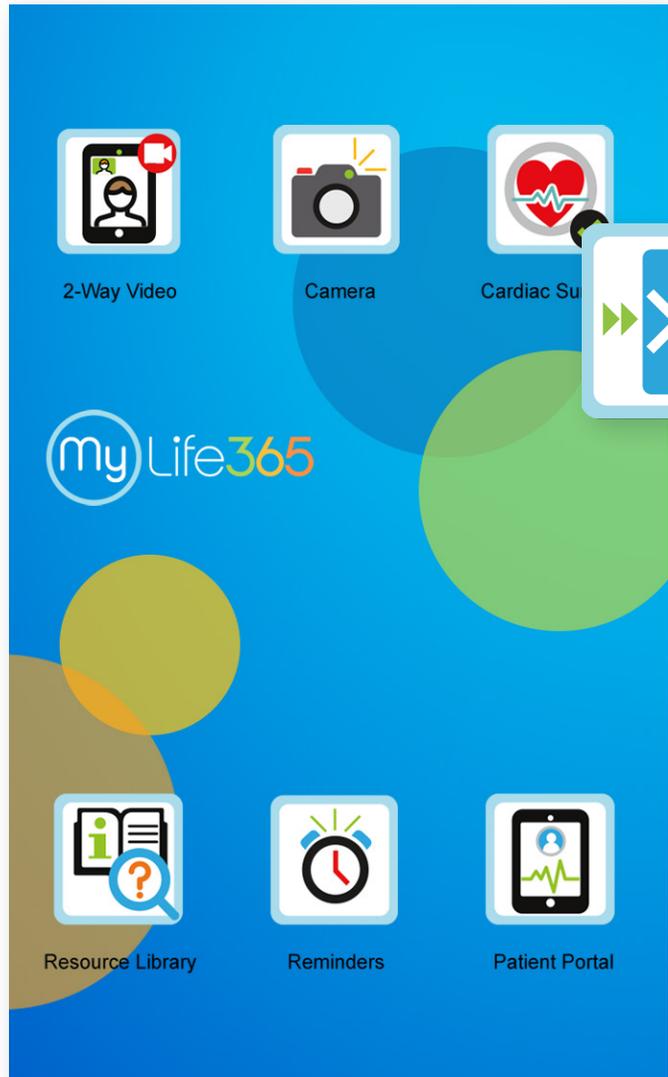

Open the Bluetooth Pairing application to pair Bluetooth Medical Devices.

**This Application's location may vary per device.*

Life365 Connect Application: **Accept Prompts**

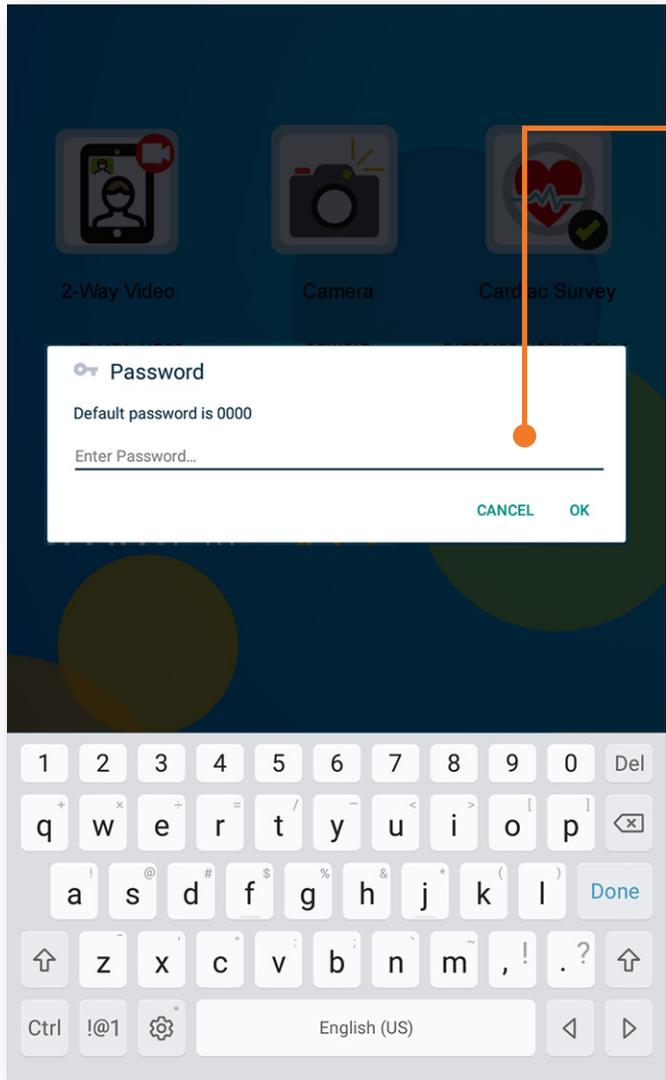

Enter your organizations password to access.

Accept all prompts that display by Selecting **Allow**.

 **Allow Life365 Connect to make and manage phone calls?**

1 of 3

 **Allow Life365 Connect to access this device's location?**



2 of 3

 **Allow Life365 Connect to access photos, media, and files on your device?**

3 of 3



DENY **ALLOW**

Life365 Connect Application: My Devices

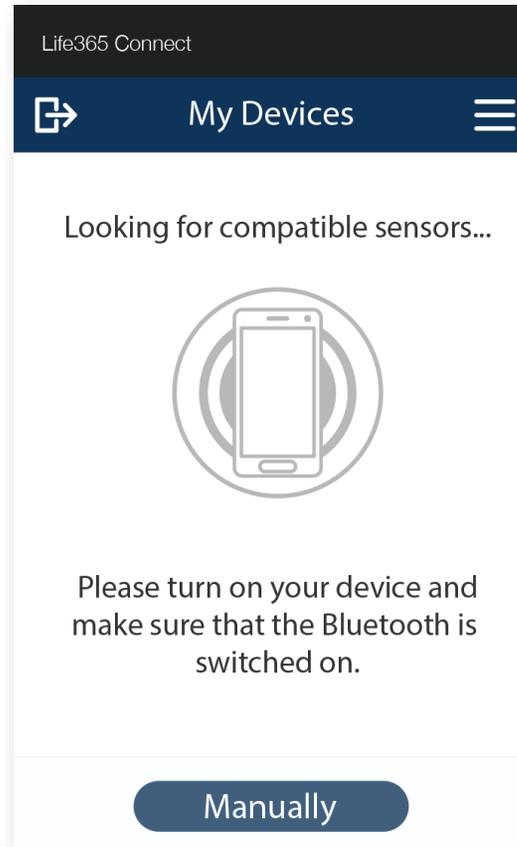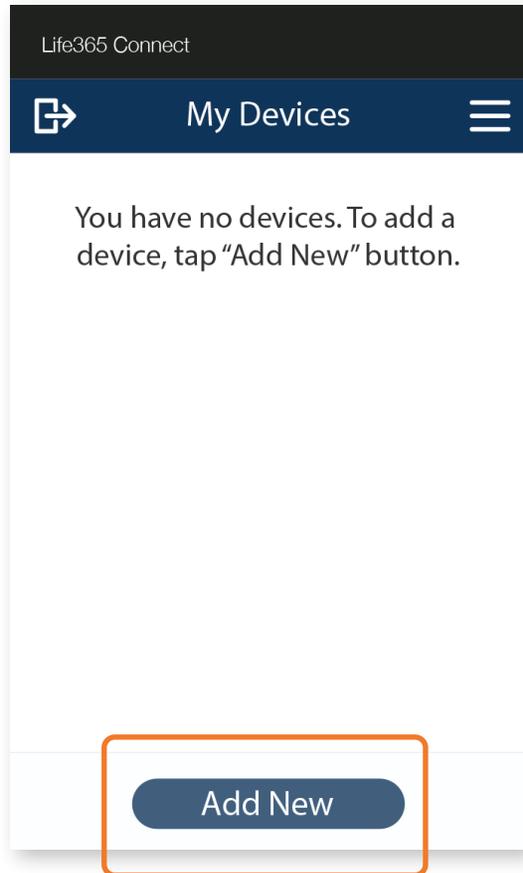

- Select **Add New** to add a Bluetooth Medical Device.
- The Smart Device will search for the Bluetooth Device.

Life365 Connect Application: My Devices (Cont.)

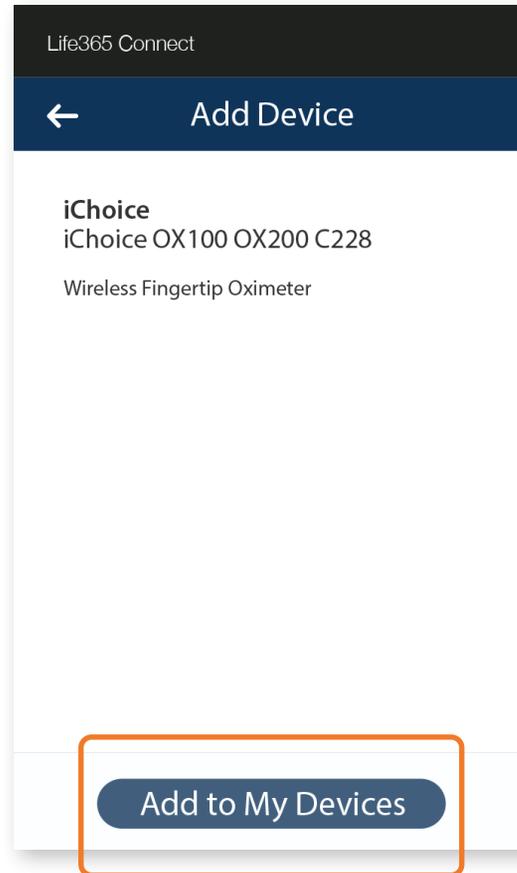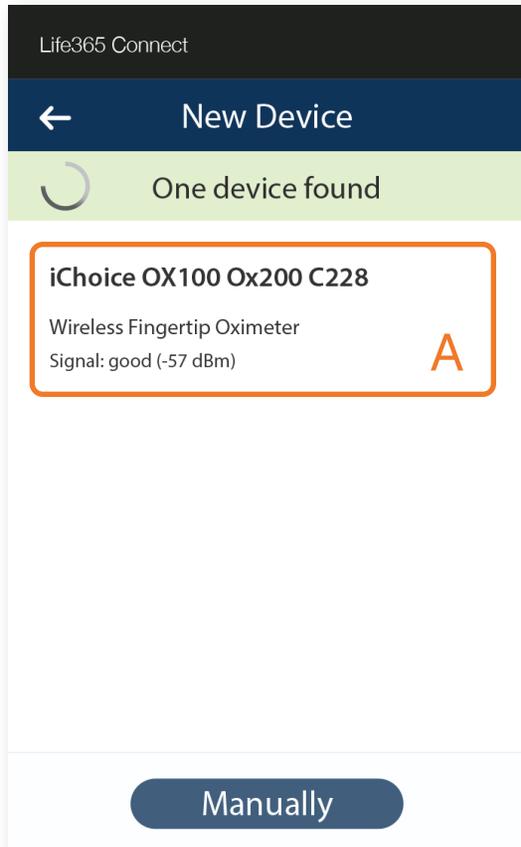


Select the device when found **(A)**.

Select Add to My Devices **(B)**.

NOTE: Some Bluetooth Devices will require the Member to complete a Vitals reading. Please refer to the manufacturers instructions on how to use device(s).

Life365 Connect Application: **Error Screen**

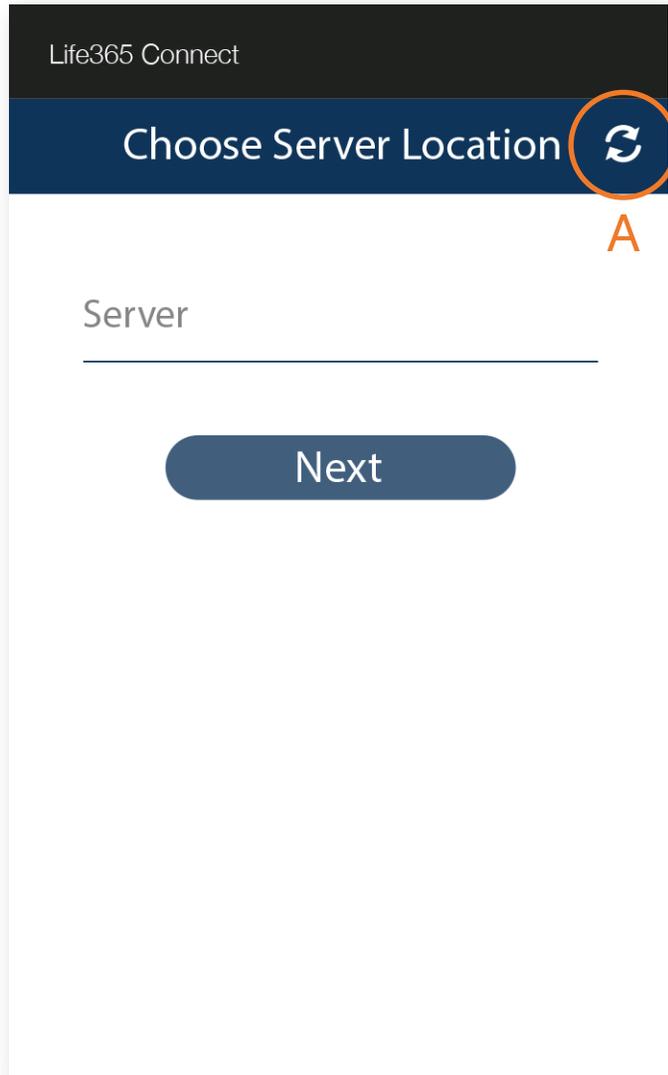



NOTE:

Always open the Reminders application before accessing the Bluetooth Pairing application. If you have opened the Bluetooth Pairing application before opening Reminders, please complete the following:

- 1 Exit the Bluetooth Application
- 2 Enter the Reminders Application
- 3 Accept any prompts that may display
- 4 Re-Enter the Bluetooth Application
- 5 Select the Refresh Symbol (A).

Accept any prompts that may display within the Bluetooth application.

Life365 Connect Application: Manual Pairing

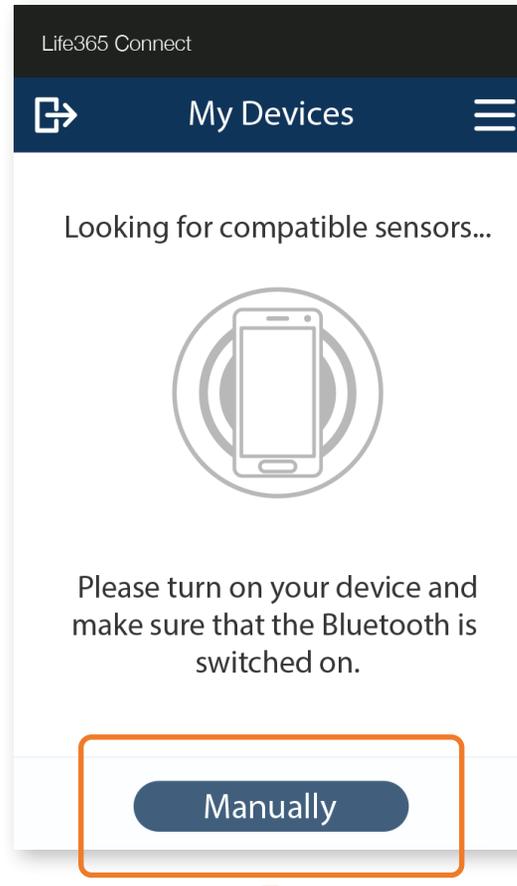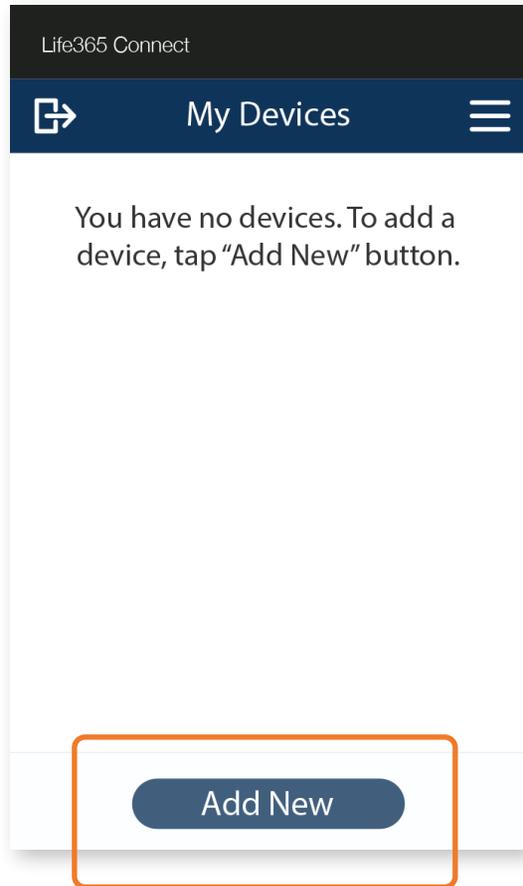


Difficulty Pairing a Bluetooth Device? Try manually pairing.

Select Add New **(A)**.

Select Manually **(B)**.

Life365 Connect Application: **Manual Pairing** (cont.)

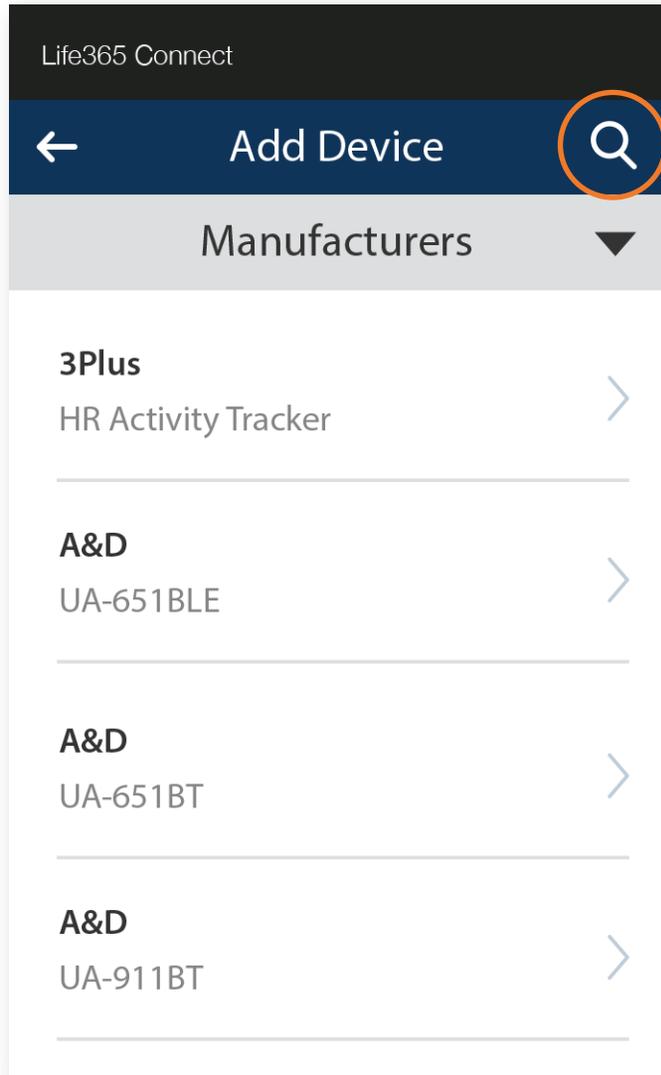

Search for the Bluetooth Device by Selecting the Magnifying glass.

Enter the Bluetooth device manufacturer.

Once found, Select the Bluetooth device.
Select **Add to my Devices**.

NOTE: *Some Bluetooth Devices will require the Member to complete a Vitals reading. Please refer to the manufacturers instructions on how to use device(s).*