

Bifocal Center Zone Starting Points Guide

	Center Zone Starting Points					
	Enhanced Monovision		Early Pres. Adds Up To +1.25		Adds Over +1.25	
	Dominant	Non-Dominant	Dominant	Non-Dominant	Dominant	Non-Dominant
	Near Center	Distance Center	Distance Center	Distance Center	Near Center	Near Center
large pupil	2.4	2.6	3.0	2.5	2.5	3.0
med pupil	2	2.2	2.6	2.3	2.2	2.6
small pupil	1.8	2.0	2.4	2.0	1.9	2.3
	Trouble Shooting More Distance -0.2mm Dom. More Near -0.2mm Non Dom.		Trouble Shooting More Distance +0.2mm Dom. More Near -0.2mm Non Dom.		Trouble Shooting More Distance -0.2mm Dom. Even More Distance -0.2mm Non Dom. More Near +0.2mm Non Dom. Even More Near +0.2mm Dom.	

	Patients Pupil Size					
	Small		Medium		Large	
pupil	Ambient	Bright	Ambient	Bright	Ambient	Bright
light level	Ambient	Bright	Ambient	Bright	Ambient	Bright
mm	3.5	2.0	4.5	3.0	6.0	3.5

Expect 2-3 Weeks Adaptation Time