

## Custom Soft Contact Lens Handling Tips

Thank you for reviewing this information about your SpecialEyes custom soft contact lenses. As you transition into your SpecialEyes high-performance lenses, you may encounter a slight learning curve regarding the care and handling of your new lenses. The following tips will help extend the life of your soft contact lenses:

**Tip 1:** Gently rub; don't scrub. SpecialEyes lenses are not meant to be "scrubbed." A light digital rub is fine; but avoid excessive scrubbing, which can tear the lens.

**Tip 2:** When preparing to insert a contact lens, carefully pour the lens from its container into your hand. Do not use fingernails or tweezers to remove your lens from the case, as this can puncture the lens.

**Tip 3:** When removing a SpecialEyes lens from your eye, you should first apply a rewetting drop and then gently slide the lens out using the pad of your finger.

After reviewing and implementing these simple handling tips, you should be able to obtain months of great vision from your SpecialEyes custom soft contact lenses.